

POC-W152 Appearance




Front-view




Rear-view

* Please use VESA compatible mounting only.

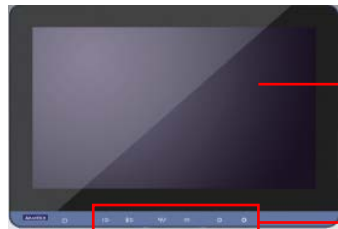

Front Bezel Screen

Function Keys

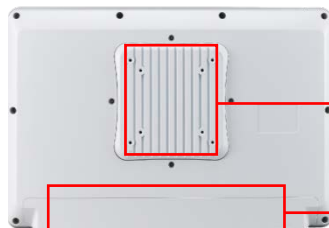

VESA Mounting

IP43 I/O Exterior

Cover



I/O Ports

Packing List


- (1) POC-W152 x 1
- (2) Accessory box x 1
 - ✓ This quick start manual x 1
 - ✓ Simpro HPU101-107 (18Vdc/5.55A) adapter x 1
 - ✓ Drivers & Manual CD x 1
 - ✓ Tool bag x 1: spare screws x 4, rubber connectors x 13, and DRAM thermal pad x 1.
 - ✓ Mylar x 1: for USB bracket fixture

Function Keys



- (1) Power
- (2) Volume Up/Down
- (3) Touchscreen Status Control
- (4) Read Light Control
- (5) Brightness Decrease/Increase

I/O Ports

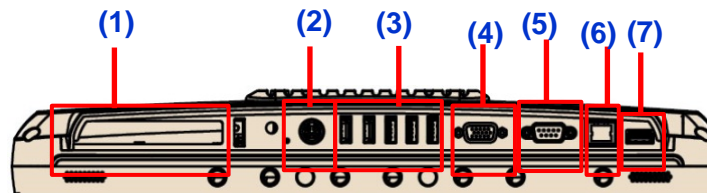

- (1) PCIe x4 Card Slot
- (2) Power DC-in
- (3) USB 1~USB 5 ports * (2 x USB 3.0, 3 x USB 2.0)
- (4) VGA Port
- (5) COM port
- (6) Gigabit Ethernet (RJ-45)
- (7) HDMI-out port

Easy Setup

Power On Computer:

- 1) Make sure Equipotential Terminal is connected to equipotential ground in the hospital.
- 2) Connect the adapter to Power DC-IN. Please note that using different adaptors may put the system into risk.
- 3) Press Power On/ Off. The button will turn green.

Driver Installation:

Advantech supports "one key" driver installation. A user can simply double click an icon to install all the drivers.

- 1) Make sure the OS has been successfully installed. If you have difficulty installing an OS, please follow the instructions in the POC-W152 User Manual.
- 2) Download the "one key" package from the Advantech Support website:
http://support.advantech.com/support/new_default.aspx
 Open the folder and double click "InstallAll.exe."
- 3) An Install dialog will appear. Follow the instructions shown in the dialog to finish the installation.

* Please use a clean OS to install this auto installation; otherwise, unexpected errors may occur.

* Should you only want to install specific drivers, please follow the instructions in the POC-W152 User Manual.

Notice:

- PCIe connection requires matching of specification. Not every PCIe card assures compatibility. Contact Advantech regional service center before use

(4) Cleaning and Disinfecting

During normal use, a POC (Point-of-Care) terminal may become dirty and should be cleaned regularly.

- 1) Prepare a cleaning agent per manufacturer's instruction or hospital protocol.
- 2) Prepare a clean cloth that has been moistened in a cleaning solution.
- 3) Wipe the POC thoroughly with a clean wipe.

Applicable Cleaning Agents List

No	Cleaning Agents	No	Cleaning Agents
1	Cidex	11	Virkon and water (1:100)
2	Isopropyl alcohol	12	DISPATCH
3	Green tinctured soap	13	SDW 70C
4	Windex	14	Trigene
5	Alcohol	15	ANTIGONE WIPES
6	Alcohol 70%	16	SporeClear

7	Incidin plus	17	AntiGone
8	Incidin liquid	18	MikroBac
9	Mikrozid liquid		
10	Chloride 1000ppm		

- * Do not immerse or rinse a POC terminal or its peripherals.
- * Do not spray cleaning agents on the chassis
- * Do not use disinfectants containing phenol

Edition 1 Part Number: 2008W15210
 Sep 2015